

## **INSTALLING INTELEVIEWER**

### **Install Software**

1. Load Intelrad CD into drive or plug in USB key
2. Go to "My computer"
3. Click on the drive with CD or USB removable device
4. Start "Install Inteleviewer.cmd" and this will start the installation.
5. Click "I Agree" for the Inteleviewer license.
6. Click "I Agree" for the Java license.
7. Keep clicking Next until finished. Do not install the DICOM service.
8. The installation process will finish and the software has been installed but needs to be configured.

### **Configure Software**

9. Start Inteleviewer (use the Inteleviewer icon on the desktop). It will display a message saying that it needs a server bookmark. Click OK.
10. Enter <https://viewer.pacificradiology.co.nz>
11. Tick the custom name box and enter Pacific Radiology.
12. Click OK and then close Inteleviewer.
13. Restart Inteleviewer and login using your username and password.

Call the PACS Help Desk on 04-978-5585 if you need any assistance.